



# Shared Health – Emergency Response Service

General - Safe Work Procedure


## Respirator Mask Doffing MSA Advantage 200LS



Hazards Present:	Personal Protective Equipment:	Minimum Training Requirement:
Biohazard	None	N/A
Purpose:	Indications:	
To provide personal protection from potentially contaminated surfaces of respirator	Respirator no longer required	
Equipment:	Contraindications:	
MSA Advantage 200 LS Respirator	None	

### Safe Work Procedure

1. Observe *Routine Practices* and use *proper body mechanics*
2. Perform hand hygiene using an alcohol based hand sanitizer
3. Loosen the neck strap and undo the clasp
4. With head tilted forward, slide the cradle headband forward over your head while maintaining control of respirator
5. Remove respirator from face
6. Perform hand hygiene using an alcohol based hand sanitizer
7. Clean the face mask, following the Safe Work Procedure - Cleaning MSA Advantage 200 LS Respirator Mask

Steps for Doffing MSA Advantage Respirator Half Mask	Important Points
1. Observe Routine Practices and use proper body mechanics	Following Routine Practices can reduce the transmission of pathogens and other microorganisms
2. Perform Hand Hygiene 	Use and alcohol based hand sanitizer



## Respirator Mask Doffing MSA Advantage 200LS



3. Loosen the neck strap and undo the clasp.



4. With head tilted slightly forward, slide the cradle headband forward over your head while maintaining control of respirator.



5. Remove respirator from face.



6. Perform Hand Hygiene



Use an alcohol based hand sanitizer



# Shared Health – Emergency Response Service

General - Safe Work Procedure

## Respirator Mask Doffing MSA Advantage 200LS



Shared health  
Soins communs  
Manitoba

7. Follow Safe Work Procedure - <b>Cleaning MSA Advantage 200 LS Respirator Mask</b>	
<b>Guiding Documentation</b>	Created with assistance from WFPS
MSA Advantage 200 LS Respirator Facepiece Instruction Manual	Training Reviewed: April 3, 2020
	IP&C reviewed: 04/28/2020
	OESH reviewed: 04/28/2020
	Approved by : 04/28/2020
	Implemented: 04/29/2020
Revised:	